**Experience Relief.
Discover Sleep.**

Operation Manual Battery

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IMPORTANT

This is a prescription-only device.

Read this operation manual before you use your Relaxis. Refer to Table 4 for an explanation of the symbols used.

CONTENTS

- One Relaxis® battery vibratory counterstimulation device (includes power supply P/N 06-0026-01)

PURPOSE OF THE DEVICE (INDICATIONS FOR USE)

The purpose of Relaxis is to improve quality of sleep in patients with primary Restless Legs Syndrome (RLS) through the use of vibratory counterstimulation.

WHEN THE DEVICE SHOULD NOT BE USED (CONTRAINDICATIONS)

Relaxis should not be used by patients who have been diagnosed with deep venous thrombosis (DVT) in either leg during the last 6 months because of the known potential to dislodge or break up the clot.

If you have questions about when you should not use Relaxis, please ask your doctor.

WHAT ARE THE POTENTIAL BENEFITS OF USING RELAXIS?

Relaxis may help improve overall quality of sleep for people with primary RLS. It is not intended to treat RLS or improve any other symptoms of RLS. It may take up to 4 weeks for you to see an improvement.

Relaxis was tested on men and women with primary RLS. During the study, some people used Relaxis and some people used a different device that did not vibrate. All participants answered questions about their quality of sleep. They answered these questions at the start of the study and after using the device for 4 weeks. Results showed that the people who used Relaxis were more likely to report less sleep disturbance and better sleep quality than those who used the other device. The results did not show a difference between the two devices for any other RLS symptoms, patients' quality of life, or patients' RLS, in general.

If you have questions about the potential benefits of Relaxis, please ask your doctor.

WHAT ARE THE POTENTIAL RISKS WITH USING RELAXIS?

Relaxis may cause your RLS symptoms to worsen. It may also cause leg cramping, tingling, soreness, pain, or motion sickness.

The most common problem reported during the study was worsening of RLS symptoms. However, this worsening resolved within 3 weeks of stopping use of Relaxis. In addition, the symptoms returned to the same level reported before using Relaxis. There were no long-term effects seen. Leg cramping, tingling, soreness, pain, and motion sickness were also reported, but much less often. All of these problems resolved on their own without any other treatment.

If you have questions about the potential risks with using Relaxis, please ask your doctor.

WHAT ARE THE ALTERNATIVES TO USING RELAXIS?

There are other alternatives to treat the symptoms of RLS. The standard way of treating RLS is with drugs. Ask your doctor what treatment may be best for you.

WARNINGS AND CAUTIONS

A *warning* means that there is a possibility of injury to the user. A *caution* means that there is a possibility of damage to the device.

! If you are currently taking medication(s) for RLS, continue to take them as prescribed until your doctor instructs otherwise. Consult with your doctor if you wish to change or stop your medication(s).

! Relaxis is only intended for use by patients with primary Restless Legs Syndrome (RLS). It is not intended for use by people who have:

- a) Leg skin disorders, such as eczema, psoriasis, cellulitis, and non-healing wounds; or
- b) Secondary RLS.

! Contact your doctor if you have secondary RLS caused by iron deficiency anemia, back pain, kidney failure, diabetes, peripheral neuropathy (numbness or tingling in hands or feet), or pregnancy. Relaxis has not been tested on people with secondary RLS, nor is it intended for use by people with secondary RLS.

! Relaxis has not been tested on people who are taking drugs that may cause RLS or RLS symptoms. Safety and effectiveness have not been established in these people.

! Only apply Relaxis to normal, intact, clean, healthy skin.

! You should be physically able to remove Relaxis if necessary.

! You should setup and use your Relaxis only as described in this operation manual.

- ! Keep your Relaxis away from children. It may contain small parts that pose a choking hazard. Strangulation could occur if children become entangled in the device cables.
- ! Do not spill fluids on the power supply. This could result in a shock hazard or Relaxis device failure. Avoid spilling fluids on the pad and controller. This could result in device failure.
- ! Keep wireless communications equipment at least 3 feet (.91 meters) from your Relaxis. This includes wireless home network devices, mobile phones, cordless telephones and their base stations, and walkie-talkies. This equipment could affect your Relaxis.
- ! Only clean your Relaxis as described in the Cleaning section.
- ! Do not try to fix or modify your Relaxis. This may be unsafe. Follow the instructions in the Troubleshooting section as needed. Contact your distributor if you need additional help.

PRODUCT DESCRIPTION

Your Relaxis consists of a pad, controller, and power supply (Figure 1). The pad is designed so that you can activate it (using the controller) at the onset of an RLS episode while lying in bed. You can place the pad under or between your legs and use the controller to start or stop a therapy cycle. During a cycle, you can also use the controller to adjust vibration intensity at any time.

The controller contains a rechargeable battery that lets you use Relaxis without plugging the power supply into a wall outlet. The life and performance of the battery over time depends upon care and storage of your Relaxis. (Read the Battery Upkeep section for details on how to maximize the life of the battery.) On “full charge,” the battery can power your Relaxis for up to two back-to-back therapy cycles. After two cycles, you will need to recharge the battery. You can plug the power supply into any standard wall socket (120VAC). It powers your Relaxis while you use it and recharges the battery.

Once turned on, Relaxis runs a 35-minute therapy cycle. During the first 30 minutes, the pad provides vibratory counterstimulation at the vibration intensity you choose. Then, during the last 5 minutes of the 35-minute cycle, Relaxis slowly reduces vibration intensity and quietly shuts off on its own.

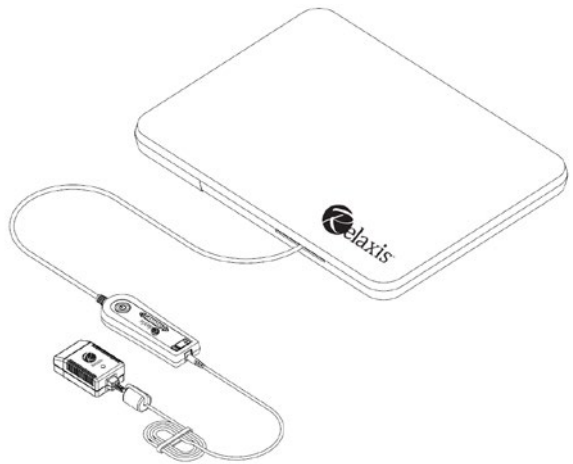
Figure 1: Relaxis device

RELAXIS COMPONENTS

Your Relaxis consists of a pad, controller, and power supply (Table 1).

TABLE 1: EXPLANATION OF COMPONENTS

PAD

The pad has vibration sources built into the foam to produce vibration. The foam is covered with a water resistant cover. You cannot remove or wash this cover. The outermost cover that is included with your device is made of cotton. You can remove and wash this outer cover.

Warning: The pad will become warm when you use your Relaxis device. During testing, the pad temperature reached a high of 42°C (108°F). (Refer to the Technical Information section for details.) Stop use of Relaxis if the heat causes discomfort or pain.

Warning: People with known allergic reactions to the pad materials should not use Relaxis. (The Technical Information section lists the pad materials.) If a skin reaction occurs after you use Relaxis, contact your doctor.

Note: Contact your distributor if you need to purchase a replacement outer (cotton) cover.

CONTROLLER

The controller has the Power Button to start/stop a therapy cycle. The two arrow buttons allow you to adjust vibration intensity while you use your Relaxis. The controller also contains a rechargeable battery.

Important: Read the Battery Upkeep section to learn how to maximize the life and performance of the battery.

Caution: Do not try to replace or repair the battery within the controller.

Warning: The controller will become warm during use. During testing, the highest temperature seen was 48°C (119°F). (Refer to the Technical Information section for details.) Avoid more than 1 minute of contact with the controller as a precaution.


Warning: People with known allergic reactions to the controller materials should not use Relaxis. (The Technical Information section lists the controller materials.) If a skin reaction occurs after you use Relaxis, contact your doctor.

POWER SUPPLY

You can plug the power supply into any standard wall socket.

Warning: Only use your Relaxis device with the power supply provided (P/N 06-0026-01). Do not connect any other power supplies or equipment to your Relaxis. This may be unsafe.

Note: Contact your distributor if you need to purchase a replacement power supply.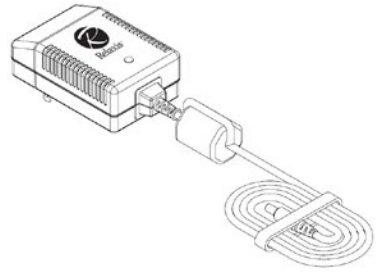

DIRECTIONS FOR USE

If you need help setting-up or using your Relaxis, contact your distributor.

1. To remove your Relaxis device from the packaging, slide the controller and power supply out from the cardboard sleeve as indicated by the arrow in Figure 2. Then, remove the pad, controller, and power supply from the box. Carefully remove any protective wrapping to avoid device damage.

Caution: If packaging has been damaged, check that your Relaxis works properly.

Caution: Do not carry the pad by the cord, and do not use the power cord to pull the power supply from the wall socket. This could damage the cables. Do not use your Relaxis if the cables are damaged.

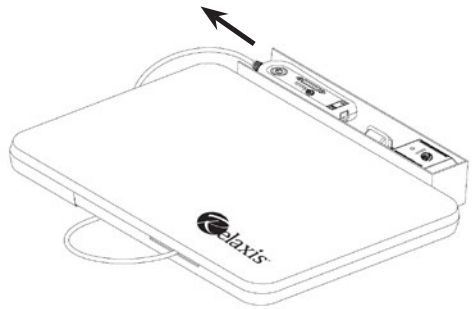
Figure 2: Packaged device

2. Plug the power supply into the controller (Figure 3).

Caution: Do not bend the controller cord or power cord excessively. This may cause damage to the device.

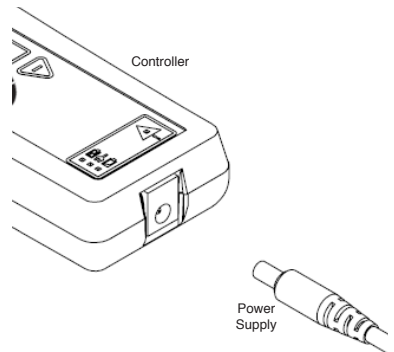
Figure 3: Connecting the power supply to the controller

3. Plug the power supply into a wall socket. You may use an extension cord if needed. Once you plug in your Relaxis device, the green light on the power supply will turn on (Figure 4). The green Line Power Light on the controller will also turn on (Figures 5 and 6).

Note: If the green light on the power supply does not turn on (Figure 4), refer to the Troubleshooting section.

Note: If the yellow Fault Light (Figures 5 and 6) turns on at any time, refer to the Troubleshooting section.

Warning: Only plug the power supply into a standard wall outlet.

Warning: Do not use your Relaxis outside of the temperature range stated in the Technical Information section.

4. The battery in the controller will charge while your Relaxis device is plugged into the wall socket. It will also charge while you use your Relaxis with the power supply plugged into the wall socket. The green Battery Charging Light (Figures 5 and 6) will turn on when the battery is charging. This light will turn off when the battery is fully charged.

To use your Relaxis on battery power only, make sure that you have fully charged the battery before you unplug the power supply from the controller.

When the battery is low, the yellow Low Battery Light will turn on (Figures 5 and 6).

To recharge, plug the power supply into the controller (step 2). Then, plug the power supply into a wall socket (step 3).

Important: Read the Battery Upkeep section to learn how to maximize the life and performance of the battery.

5. Gather any excess cable length and secure it with the Velcro strap provided. Position cables to avoid strangulation and tripping.

Warning: Never use your Relaxis if the device or cables are damaged.

Warning: Use caution when stepping over or around cables to prevent tripping.

Warning: Minimize excess cable length to avoid strangulation.

6. Place the pad in the desired location (e.g. bed). Use the side of the pad that will allow you to access the controller easily while you use your Relaxis. You can use either side of the pad.

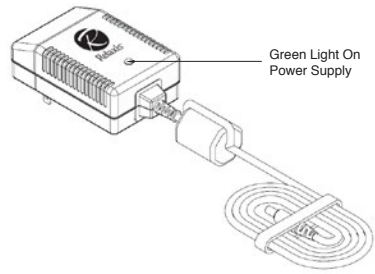
Figure 4: Power supply light

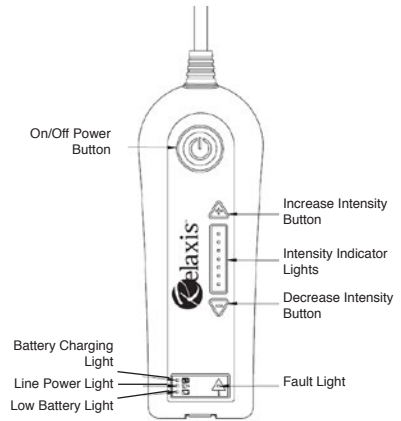
Figure 5: User controls and lights

Figure 6: Lights at the bottom of controller

- Lie down and put the pad under or between your legs or calves. You should put the pad where your RLS symptoms occur (Figures 7 and 8).

Caution: Do not bend or wrap the pad around your legs. This could damage the pad. The pad should lay flat.

- Press the Power Button on the controller to start a 35-minute therapy cycle (Figure 5).

Note: To stop a therapy cycle at any time, push the Power Button.

Note: To disconnect your Relaxis from power, remove the plug from the wall socket.

- Use the two Intensity Buttons on the controller to adjust vibration intensity (Figures 5 and 9). These buttons are arrow-shaped. The button marked with a “+” allows you to increase intensity. The button marked with a “-” allows you to decrease intensity.

Each time you press an arrow-shaped button, the closest Intensity Indicator Light will blink. This shows that you have changed the intensity of vibration. If you press an arrow-shaped button many times, an Intensity Indicator Light will either light up or go out. It will light up to show that you have increased the overall intensity. It will go out to show that you have decreased the overall intensity.

If you wish to quickly increase or decrease the level of intensity, press and hold down an arrow-shaped button. Release the button when you reach the level of intensity you want. Use the intensity level that feels best for you. Lower the intensity if your symptoms start to feel worse.

Note: Each time you turn on Relaxis, it will automatically start at the vibration intensity you set during the last cycle. It is always okay to adjust the vibration intensity so that it feels comfortable for you.

Note: You may feel a single pulse of heightened vibration intensity when you first turn on Relaxis. This may occur if the device turns on at low vibration intensity. This is normal.

Warning: Stop using Relaxis and contact your doctor if your RLS symptoms worsen.

- During the last 5 minutes of the therapy cycle, Relaxis will slowly reduce vibration intensity and quietly shut off on its own.

Figure 7: Pad position example for sleeping on your back

Figure 8: Pad position example for sleeping on your side

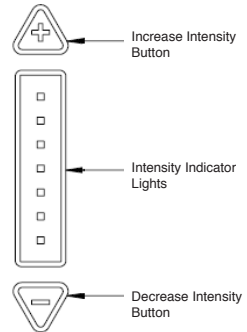
Figure 9: Controls for adjusting vibration intensity

11. To start another cycle, press the Power Button.

Warning: You should not initiate more than two back-to-back 35-minute therapy cycles. Testing has not been performed to support more than two back-to-back therapy cycles.

Warning: Only use Relaxis as needed. Too much stimulation may lead to discomfort.

TROUBLESHOOTING

Note: Please contact your doctor if you experience a negative outcome or negative effect while using Relaxis. This may include lack of sleep quality improvement or known clinical adverse effects such as worsening of RLS symptoms, leg cramping, tingling, soreness, pain, or motion sickness. If you experience these or any other problems when using the device – such as wounds, abrasions, or a Relaxis that does not work properly – please contact your distributor or Relaxis (call 888-475-7435).

In the US you can also report negative outcomes or effects to the FDA through MedWatch. (Call 1-800-332-1088 or go online to <https://www.accessdata.fda.gov/scripts/medwatch/>.) In Canada, you may also report negative effects to Health Canada. (Go online to www.healthcanada.gc.ca/medeffect/ and follow directions for reporting medical device related incidents by consumers.)

TABLE 2: TROUBLESHOOTING

There are no parts that can be fixed, modified, or replaced by the user. Contact your distributor if a problem remains unresolved after following the steps in the Troubleshooting section. You should also contact your distributor if any other problems occur.

TABLE 2: TROUBLESHOOTING

Problem	Possible Solution
1. No power (green light on the power supply is not on)	<ol style="list-style-type: none"> 1. If the green light on the power supply is not on, there is no power going to the power supply. 2. Make sure the plug is pushed all the way onto the power supply. 3. Make sure the power supply is securely plugged into the wall outlet. 4. Confirm you are using a “live” outlet. Plug the power supply into a different outlet and check if the green light on the power supply turns on.
2. No power (green Line Power Light does not turn on)	<ol style="list-style-type: none"> 1. Make sure the green light on the power supply is on. If it is not, follow steps in Problem 1. 2. Make sure the power supply is plugged securely into both the controller and the wall outlet. 3. Confirm that you are using the power supply provided with your Relaxis. 4. Confirm that you are using a “live” outlet. 5. Push the Power Button to turn on the device.
3. Fault Light activated (yellow Fault Light is on)	<ol style="list-style-type: none"> 1. Push the Power Button to turn off the device. 2. Wait for 15 minutes. 3. Push the Power Button to turn the device back on.

Problem	Possible Solution
4. Pad continues to vibrate after pressing the Power Button to turn off the device)	<ol style="list-style-type: none"> 1. Push the Power Button again to turn off the device. 2. If the pad continues to vibrate, unplug the power supply from the wall socket (if it is not already) and allow the device to operate until it runs completely out of battery power.
5. Relaxis device does not work properly on battery power	<p>Make sure that the battery is fully charged:*</p> <ol style="list-style-type: none"> 1. Plug the power supply into the controller and into the wall outlet. 2. Confirm that you are using the power supply provided with your Relaxis. 3. Confirm that both the green Line Power Light and the green Battery Charging Light turn on. 4. Let the battery charge until both the green Battery Charging Light and the yellow Low Battery Light turn off. 5. Unplug the controller from the power supply. 6. Push the Power Button to turn on the device. <p><i>* Note: If you have not charged the battery within the last 6 months, the battery may not be able to charge. However, your Relaxis will still work while plugged into a wall outlet.</i></p>
6. Device shuts off before the end of the 35-minute therapy cycle	<ol style="list-style-type: none"> 1. a) If the device was using battery power when it shut off, check that the battery is fully charged (see Problem 5). b) If the device was using line power when it shut off, follow the steps below: 2. Check that the power supply is plugged securely into the controller and the wall outlet. 3. Confirm that you are using the power supply provided with your Relaxis. 4. Push the Power Button to turn on the device. 5. Confirm that the green Line Power Light turns on. 6. Adjust the vibration intensity to the setting that feels best for you.
7. Cannot change the vibration intensity	<ol style="list-style-type: none"> 1. Push the Power Button to turn off the device. 2. Push the Power Button to turn the device back on. 3. Hold down the Increase Intensity Button until all of the Intensity Indicator Lights turn on. 4. Adjust the vibration intensity to the setting that feels best for you.

BATTERY UPKEEP

The life and performance of the battery depends upon storage temperatures and charging upkeep.

- Store your Relaxis at room temperature to maximize battery life. Elevated temperatures during use and storage degrade the life and performance of the battery.
- You should recharge the battery occasionally (at least once every 6 months). Otherwise, the battery may lose its ability to recharge.

USE, TRANSPORT, AND STORAGE CONDITIONS

Your Relaxis is designed for use at home when you experience an RLS episode in bed. It is important to protect your Relaxis from extreme temperatures and humidity during use, transport, and storage. See the Technical Information section for details on allowable temperatures and conditions for use, transport, and storage of your Relaxis device.

Warning: Do not use your Relaxis in moist, wet, or dusty areas. This increases the risk of electrical shock and device failure.

Warning: Do not use your Relaxis in oxygen rich areas. It is intended for use in places with normal air.

Caution: Keep your Relaxis away from children as well as pets and pests to avoid accidental device damage.

CLEANING

Caution: Only the outer pad cover and device surfaces may be cleaned. The inner pad cover is not removable. Do not try to remove the inner cover from the pad.

Caution: Do not attempt to remove the controller cord from the pad. It is not removable.

Caution: Never immerse your Relaxis device in water or use cleaning solutions on device surfaces.

To clean your Relaxis:

1. Unplug the device before removing the outer cover from the pad.
2. If needed, remove the outer (cotton) cover and hand or machine wash. To machine wash, turn the cover inside out and wash it in cold water with like colors. Dry the cover on a low setting (with the cover inside out) or line dry.
3. If needed, wipe the controller and pad surfaces with a clean, damp (not dripping) cloth. Use only water to dampen the cloth. Do not use cleaning solutions. These could damage the device surfaces.

To put the outer (cotton) cover back on your Relaxis after washing:

1. Lay the cover flat over the pad so that the side slit in the cover aligns with the position of the controller cord.
2. Place the controller into the cover opening and then pass it through the side slit.
3. Guide the controller cable completely through the side slit.
4. Slide the cover over the pad.

DISPOSAL

Because your Relaxis has electronic parts, it is electronics waste. Follow your local, state, and national regulations for electronics waste when disposing your Relaxis.

Warning: Your Relaxis device contains a lithium-ion battery. Do not dispose of the battery in fire.

TECHNICAL INFORMATION

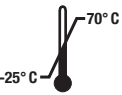
The Relaxis battery device is a stand-alone device with no accessories.

TABLE 3: TECHNICAL INFORMATION

EXPECTED SERVICE LIFE:	3 years
PAD MODEL NUMBERS:	09-0021-01 (Small, 15" x 17") 09-0021-02 (Medium, 16" x 17") 09-0021-03 (Large, 17" x 23")
PAD MATERIALS:	Cotton Polyvinyl chloride (PVC) coated Nylon Polyurethane foam
CONTROLLER MATERIALS:	Acrylonitrile butadiene styrene (ABS) Polyester plastic
BATTERY (WITHIN CONTROLLER):	Rechargeable lithium-ion battery
POWER SUPPLY:	Medical grade Class II power supply (120-220 VAC to 18VDC converter) compatible with any standard 120VAC wall socket
APPLIED PARTS:	The controller and pad are considered type BF applied parts
MAXIMUM TEMPERATURE OF PAD DURING USE:	Pad surface may reach a temperature as high as 42°C (108°F). This temperature was measured during testing at the end of two back-to-back therapy cycles with Relaxis set at the highest setting of vibration intensity.
MAXIMUM TEMPERATURE OF CONTROLLER DURING USE:	The controller may reach a temperature as high as 48°C (119°F). This temperature was measured during testing at the end of two back-to-back therapy cycles.
ALLOWABLE NORMAL USE CONDITIONS:	5°C to +40°C, relative humidity 45% to 85% non-condensing, atmospheric pressure 700hPa to 1060hPa.
ALLOWABLE TRANSPORT AND STORAGE CONDITIONS:	-20°C to +50°C, relative humidity 45% to 85% non-condensing. Exposure to upper limit not to exceed 1 week.

SYMBOLS

TABLE 4: EXPLANATION OF SYMBOLS

<p>Rx only</p>	<p>Federal (USA) law restricts this device to sale by or on the order of a physician.</p>	 <p>Follow instructions for use. It is important to read all of the Operation Manual with careful attention to warnings and cautions before using this product.</p>
	<p>Non-Sterile</p>	
	<p>Reference Number</p>	 <p>CSA-certified for Canada and U.S.</p>
	<p>Batch Code</p>	 <p>Power supply is classified as Class II</p>
	<p>Serial Number. First 4 digits specify year of manufacture.</p>	 <p>Device has type BF applied parts</p>
	<p>Connected to power. When the green Line Power Light on the controller is illuminated, the Relaxis device is receiving power from the wall socket.</p>	 <p><i>Caution:</i> If the yellow Fault Light on the controller turns on (Figures 4 and 5), refer to Troubleshooting section.</p>
	<p>Battery is charging. When the green Battery Charging Light on controller is illuminated, the battery is charging.</p>	 <p>Low battery. When the yellow Low Battery Light on the controller illuminates, battery power is low. Plug in power supply to recharge.</p>
	<p>Temperature limits for transport and storage are between -20°C to 50°C. To maximize battery life, exposure to upper limit should not exceed 1 week.</p>	 <p>Humidity limits for transport and storage are between 45% to 85% relative humidity non-condensing.</p>
<p>IP40</p>	<p>Power supply ingress protection (IP) rating. An ingress rating of 40 indicates the power supply enclosure is protected against ingress of solid particles with a diameter greater than 1mm and has no ingress protection for water.</p>	<p>Relaxis IP rating. An ingress rating of 22 indicates the pad and controller enclosures are protected against ingress of solid particles with a diameter of 12.5mm and ingress of vertically falling water drops when the enclosure is tilted up to 15°.</p> <p>IP22</p>

LIMITED WARRANTY

Relaxis warrants that the product shall be free from defects in material and workmanship for a period of 3 years from the date of purchase by the initial consumer. This warranty does not apply to the rechargeable lithium-ion battery.

If the product fails to perform in accordance with the product specifications, Relaxis will repair or replace – at its option – the defective material or part.

This warranty does not cover damage caused by accident, misuse, abuse, alteration, modification and other defects not related to material or workmanship.

Relaxis' quality assurance department or its authorized representative shall examine any device returned for defect, and Relaxis reserves the right to charge an evaluation fee for any returned device as to which no problem is found after investigation by Relaxis.

This warranty is non-transferable by unauthorized distributors of Relaxis products, and Relaxis reserves the right to charge the dealer for warranty service of failed product not purchased directly from Relaxis or authorized distributors.

Warranty claims on defective product must be made by the initial consumer.

Relaxis disclaims all liability for economic loss, loss of profits, overhead, or consequential damages that may be claimed to arise from any sale or use of this product. Some states do not allow the exclusion of limitations of incidental or consequential damages, so the above limitation may not apply to you.

The warranty is given in lieu of all other expressed warranties. In addition, any implied warranties – including warranty of merchantability or fitness for the particular purpose are limited to 3 years. Some states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you. This warranty gives you specific legal rights, and you may also have other rights, which vary from state to state.

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© 2015 Relaxis. All rights reserved